

# Quick Reference Technique Guide

## STEP 1: Approach

Volar or Dorsal.  
(Surgeon Preference)



## STEP 2: Suture Anchor Insertion

Insert suture anchor into base of the 2nd metacarpal and or the trapezoid.  
*Preferred suture for recommended Mitek G2 or Mini Mitek anchors is a # 2 fiberwire.*



## STEP 2: Remove Trapezium

Remove Trapezium.



## STEP 5: Implantation

Insert the implant into the trapezium cavity and tension the sutures placed through the holes to position the implant. Tie sutures.



## STEP 3: Implant Trial

Select the correct size implant using the available trials.




## STEP 6: Closure

Close the capsule.



# Quick Reference Technique Guide

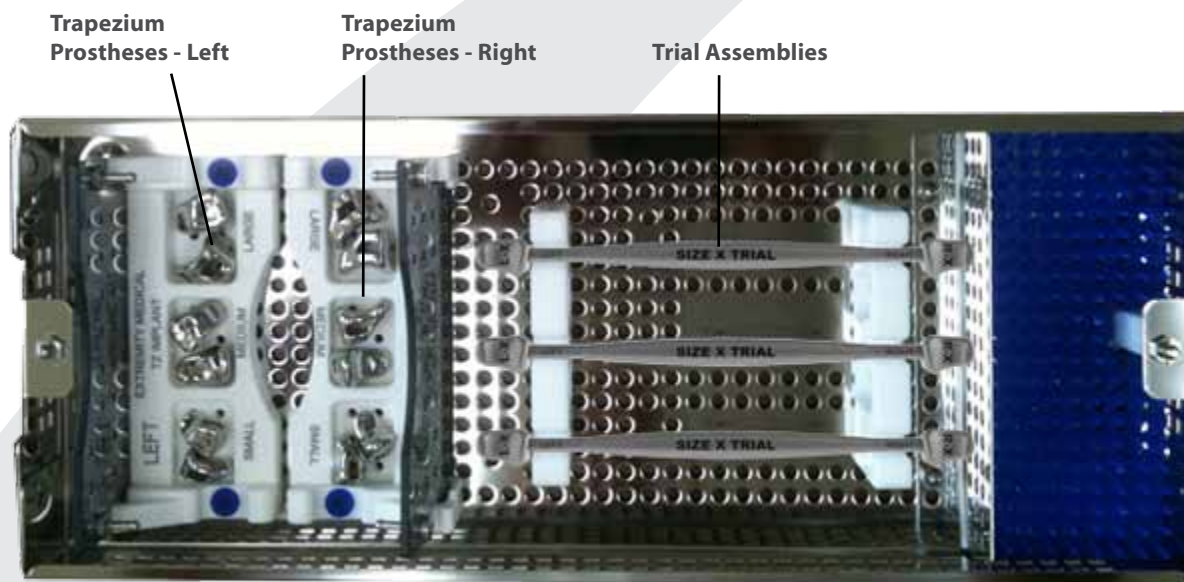

Implant Sizes	
Catalog Number	Description
103-11001	Trapezium Prosthesis (size 1) - Left
103-11002	Trapezium Prosthesis (size 1) - Right
103-21001	Trapezium Prosthesis (size 2) - Left
103-21002	Trapezium Prosthesis (size 2) - Right
103-31001	Trapezium Prosthesis (size 3) - Left
103-31002	Trapezium Prosthesis (size 3) - Right

Instruments	
Catalog Number	Description
103-00000	TrapEZ Instrument Tray
103-00010	Size 1 Trial Assembly
103-00020	Size 2 Trial Assembly
103-00030	Size 3 Trial Assembly